

Everything you need
to find the adhesive
that is **right for you**

Provox Life Adhesive Collection



Provox Life



Welcome to the Provox Life Adhesive Collection

Everyone's skin and daily needs are different, and there's no one-size-fits-all solution. That's why we've included a full selection of Provox Life Adhesives so **you can explore what works best for your skin, your lifestyle and your voice.**

The Provox Life Adhesive Collection was designed just for you – to help you find the best adhesive fit for your stoma.

The Adhesive Collection is ideal for:

- **Trying before you commit** – experience all adhesives to discover what feels right for you and find the best fit for your individual stoma shape.
- **Supporting healthy skin** – explore which adhesive and skincare routine works best with your skin type.
- **Feeling confident in your daily routine** – whether you're resting, moving, or speaking, there's an adhesive that fits.

When used with our Provox Skin Barrier wipes and Provox Adhesive Remover wipes it can help support a more comfortable overall experience.




Let's Get Started

Inside this guide, you'll find:

- A quick overview of each adhesive and when to use it.
- Helpful tips for applying and removing adhesives.
- A log to track which adhesives you used and liked most.

This is your opportunity to find the adhesive that works best for you. Take your time, try each one, and feel free to share your preferences with your clinician or Atos support team.

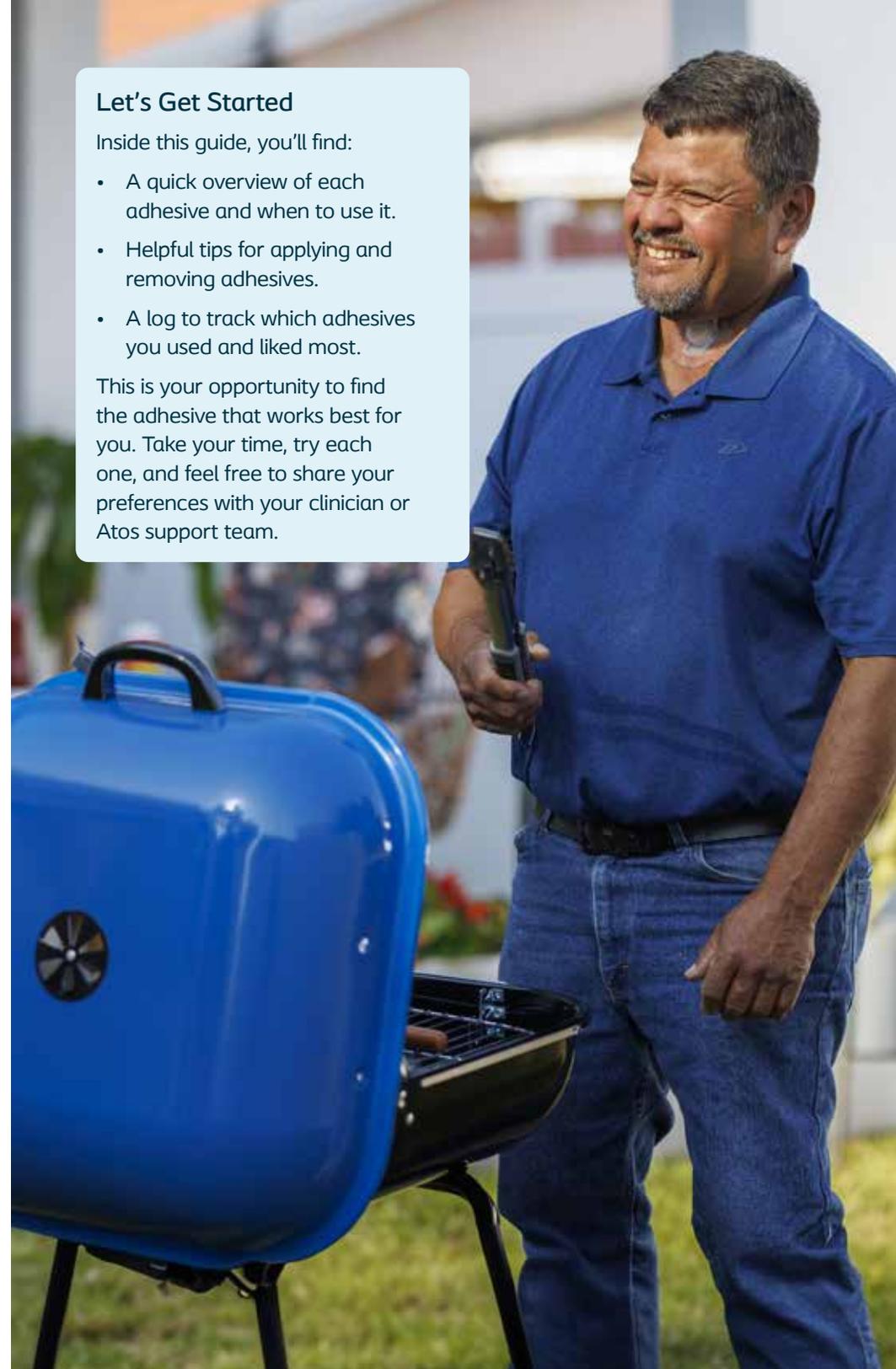

Benefits of using a Provox Life Adhesive

Wearing an adhesive allows you to comfortably attach your Heat and Moisture Exchanger (HME) over your stoma. HMEs help to restore some of your body's natural airway functions — warming and humidifying the air you breathe. This can reduce coughing, mucus production, and support better overall breathing.

There are many different Provox Life Adhesives available, and the right type and fit will depend on your individual needs. The good news is that multiple options may work well for you, allowing you to find a solution that fits your lifestyle and stoma.

Choosing the right adhesive is important because it:

- Keeps the HME securely in place.
- Creates a reliable seal around the stoma, which helps support speaking with a voice prosthesis.

When selecting the right adhesive, it's important to consider:


- The shape and depth of your stoma. Some people have deeper or uniquely shaped stomas after surgery, which can affect how the adhesive fits.
- The skin around your stoma, which can change during your recovery.
- Whether you use a voice prosthesis, which may require extra support from the adhesive to maintain a good seal for voicing.

The adhesive plays a key role by keeping the HME securely in place, helping you feel more comfortable and confident throughout your day — whether you're resting, moving, or speaking.



How to choose the right adhesive for you

There's no right or wrong place to start when trying an adhesive. The chart to the right offers simple suggestions on where to begin with your adhesive choice based on common situations.

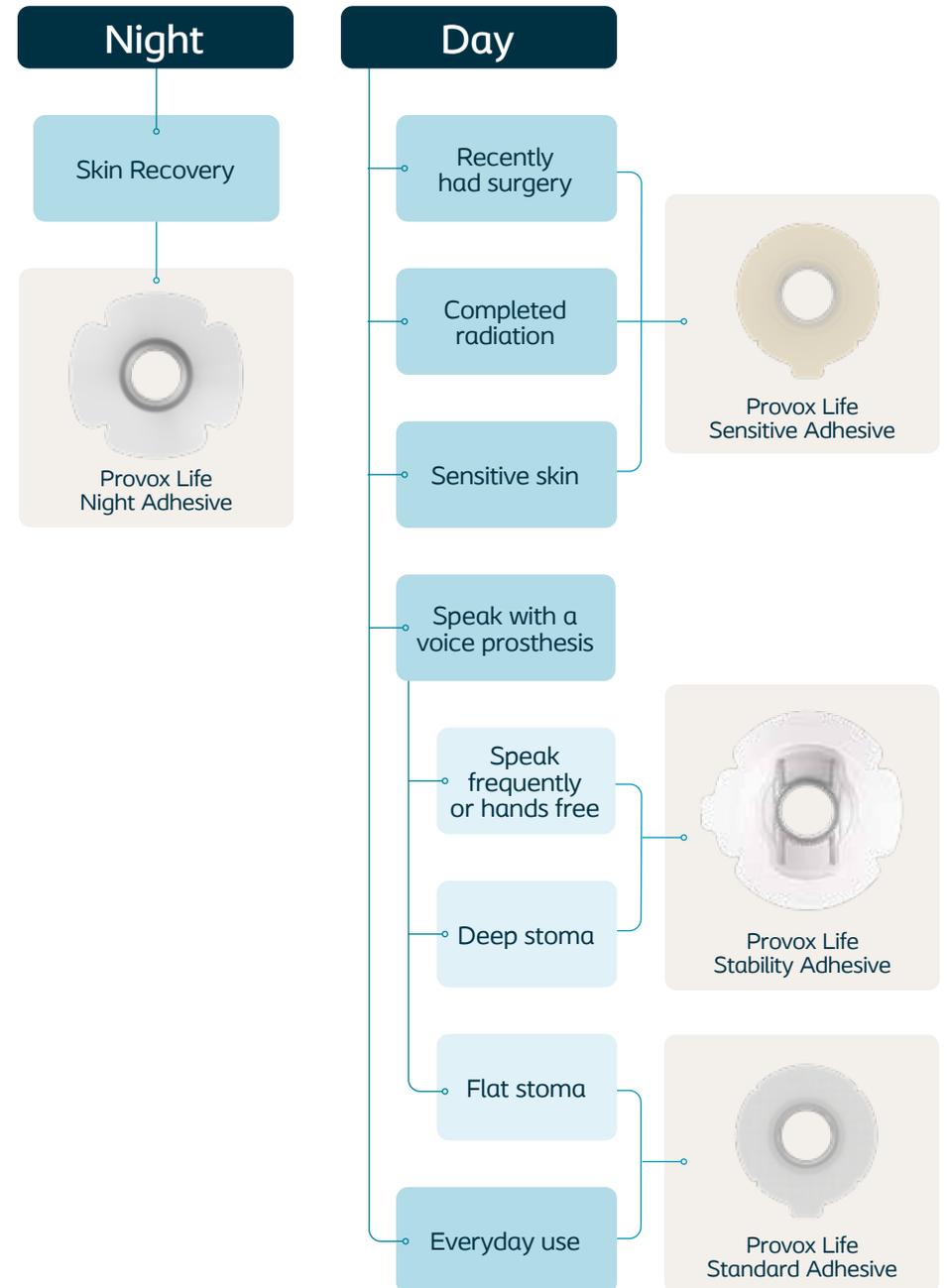

Provox Life Adhesive Collection

Provox Life Standard Adhesive



Shape	Qty
Round	4
Oval	4
Plus	4



Round


Oval

Plus

Provox Life Sensitive Adhesive



Shape	Qty
Round	4
Oval	4
Plus	4

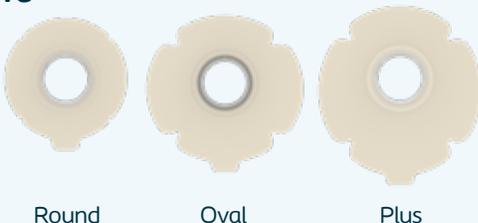

Round

Oval

Plus

Provox Life Stability Adhesive



Qty
3



Provox Life Night Adhesive



Qty
3



Healthy skin starts with a consistent routine

The skin around your stoma is delicate and can be sensitive, especially with regular adhesive changes. Using a proper skincare routine and accessories helps keep the area clean, protected, and ready for your next adhesive.

Establishing consistent habits with our Provox skincare wipes can make adhesive changes easier and more comfortable, while supporting a smoother routine and helping your adhesives perform as intended.

Provox Skin Barrier

Use after cleaning your skin to apply a thin, protective layer as a barrier between the skin and the adhesive that helps the adhesive stick to the skin.



Note: Not for use with Provox Life Night Adhesive.

Provox Adhesive Remover

Apply on top of and along the edge of your adhesive to gently remove it. Avoid getting adhesive remover in your stoma and do not forcefully pull the adhesive off as this may hurt your skin.



Note: Not for use with Provox Life Night Adhesive.

Also available

Provox Cleaning Towel

Use after taking off an adhesive and before applying a new one to help remove any oil, dirt, or adhesive residue.





Provox Life Standard


Standard Adhesive is a soft, flexible and low-profile everyday adhesive that provides a strong seal and secure fit around your stoma.



Scan here to watch a video on how to use Provox Life Standard.



www.atosmedical.us/videos/provox-life-standard-adhesive

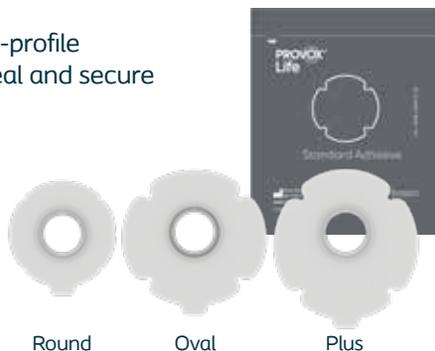

Round


Oval

Plus

How to use wipes with Provox Life Standard

Product	Clean	Protect	Remove	When to use
Provox Cleaning Towel	✓			To clean the skin before putting on an adhesive
Provox Skin Barrier		✓		To leave a protective layer on the skin
Provox Adhesive Remover			✓	To help remove the adhesive

How to apply Provox Life Standard Adhesive

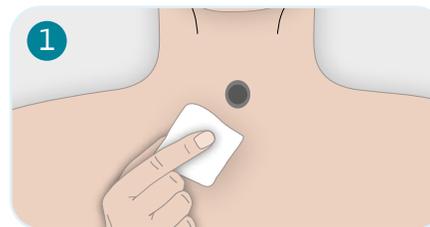


1 Clean the skin around your stoma with Provox Cleaning Towel, or with soap and water. Let it dry.




2 Apply Provox Skin Barrier (and let dry) to leave a protective layer on your skin.




3 Rub the adhesive between your hands to warm it up.

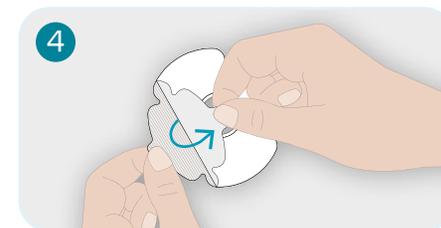

4 Remove the protective paper.

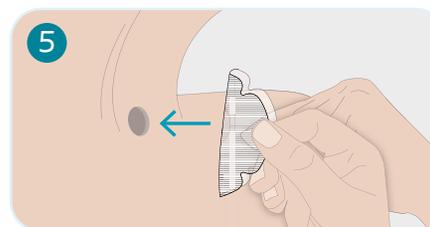


5 Apply the adhesive close to the opening of your stoma.

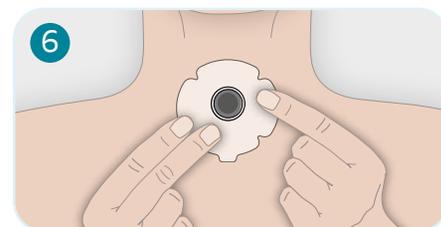

6 Massage the adhesive onto your skin. Start from your stoma and massage outwards over the entire adhesive.

How to remove Provox Life Standard Adhesive

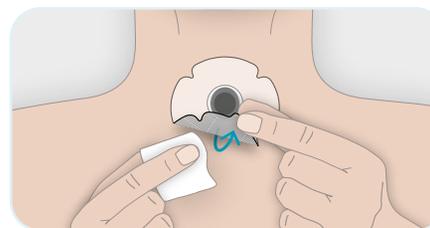

Apply Provox Adhesive Remover on top of the adhesive. Then, lift the finger tab carefully and add more underneath the adhesive. Gently remove.



Provox Life Sensitive


Sensitive Adhesive is a gentle, comfortable, and low-profile everyday adhesive for sensitive skin that forms a good seal and provides a secure fit.



Scan here to watch a video on how to use Provox Life Sensitive.



www.atosmedical.us/videos/provox-life-sensitive-adhesive

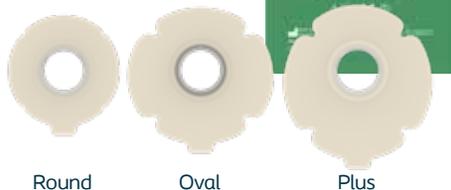

Round


Oval

Plus


How to use wipes with Provox Life Sensitive

Product	Clean	Protect	Remove	When to use
Provox Cleaning Towel	✓			To clean the skin before putting on an adhesive
Provox Skin Barrier		✓		To leave a protective layer on the skin
Provox Adhesive Remover			✓	To help remove the adhesive

How to apply Provox Life Sensitive Adhesive

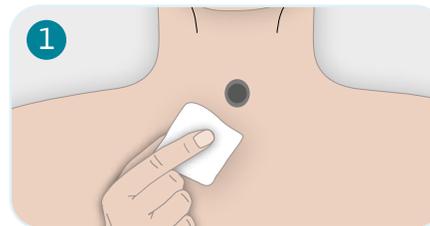


1 Clean the skin around your stoma with Provox Cleaning Towel, or with soap and water. Let it dry.

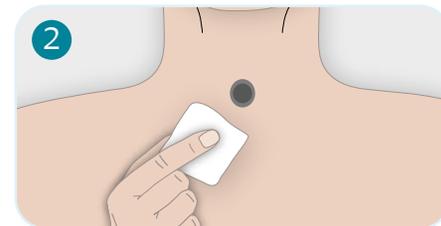


2 Apply Provox Skin Barrier and let dry. Provox Skin Barrier leaves a protective layer on your skin.




3 Rub the adhesive between your hands to warm it up.

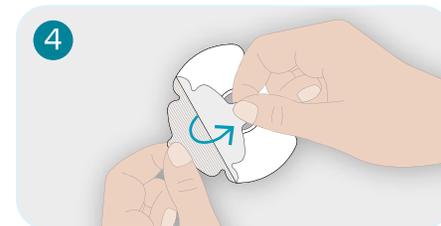

4 Remove the protective paper.

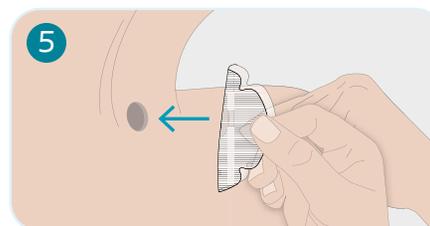


5 Apply the adhesive close to the opening of your stoma.

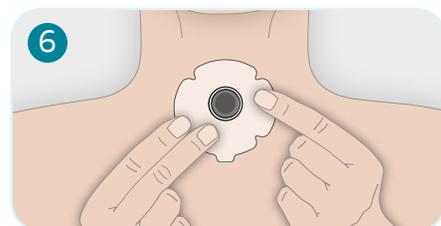

6 Massage the adhesive onto your skin. Start from your stoma and massage outwards over the entire adhesive.


How to remove Provox Life Sensitive Adhesive

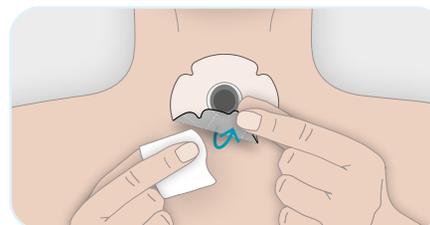


Apply Provox Adhesive Remover on top of the adhesive. Then, lift the finger tab carefully and add more underneath the adhesive. Gently remove.



How to apply Provox Life Stability Adhesive

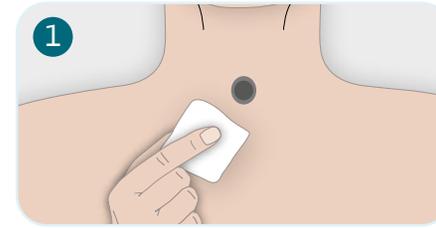


1 Clean the skin around your stoma with Provox Cleaning Towel, or with soap and water. Let it dry.

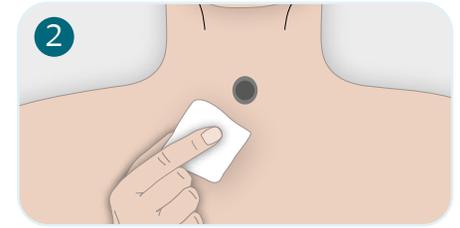


2 Apply Provox Skin Barrier and let dry. Provox Skin Barrier leaves a protective layer on your skin.




3 Rub the adhesive between your hands to warm it up.

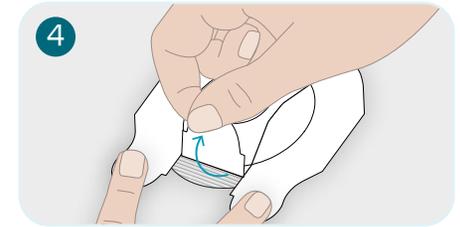

4 Remove the protective paper.

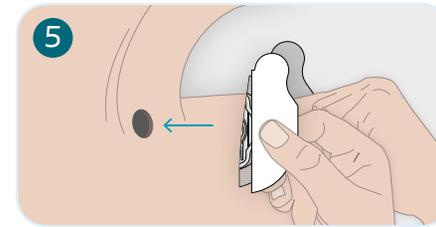

5 Apply the ring of adhesive close to the opening of your stoma.

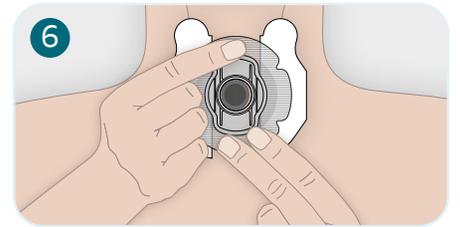

6 Massage the adhesive onto your skin. Start from your stoma and massage outwards over the entire adhesive.

Provox Life Stability

Stability Adhesive is a firm everyday adhesive that provides solidity, especially if your stoma is deep. It is an ideal adhesive for those who speak frequently or speak hands-free.



Scan here to watch a video on how to use Provox Life Stability.




www.atosmedical.us/videos/provox-life-stability-adhesive




How to use wipes with Provox Life Stability

Product	Clean	Protect	Remove	When to use
Provox Cleaning Towel	✓			To clean the skin before putting on an adhesive
Provox Skin Barrier		✓		To leave a protective layer on the skin
Provox Adhesive Remover			✓	To help remove the adhesive

How to remove Provox Life Stability Adhesive

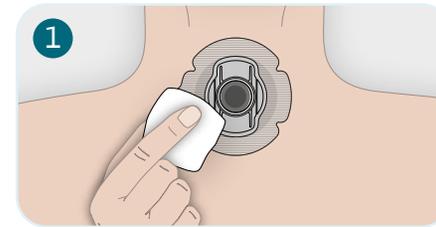

1 Use Provox Adhesive Remover for easy removal. First apply Provox Adhesive Remover on top of Stability Adhesive.

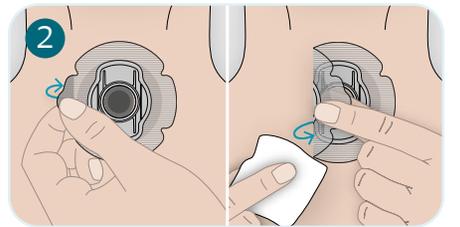

2 Then lift the finger lift tab carefully and add more underneath the adhesive. Gently remove Stability Adhesive.



Provox Life Night

Night Adhesive allows your skin to recover while you sleep. A skin-friendly hydrogel adhesive that soothes and cools the skin.



Scan here to watch a video on how to use Provox Life Night.




www.atosmedical.us/videos/provox-life-night-adhesive




How to use wipes with Provox Life Night

Product	Clean	Protect	Remove	When to use
Provox Cleaning Towel	✓			To clean the skin before putting on an adhesive


How to apply Provox Life Night Adhesive

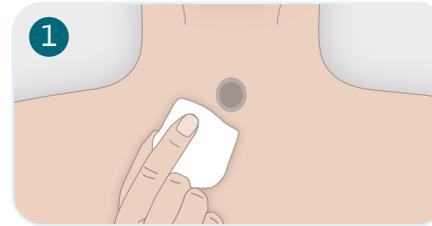


1 Clean the skin around your stoma with Provox Cleaning Towel, or with soap and water. Let it dry.

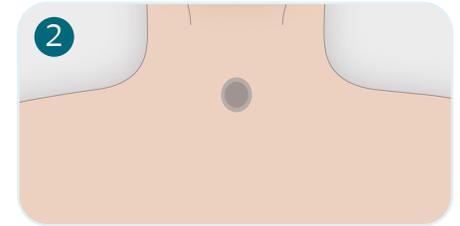


2 Let your skin dry. Do not use Provox Skin Barrier or products such as skin cream that leave an oily residue, as they may prevent the adhesive from sticking properly.

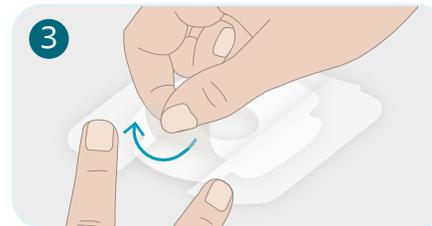

3 Remove the middle protective liner.

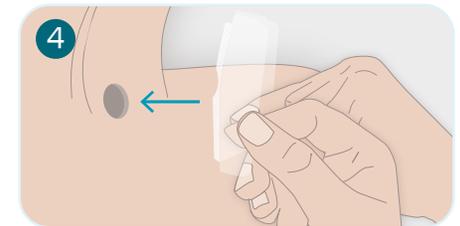

4 Apply the adhesive close to the stoma

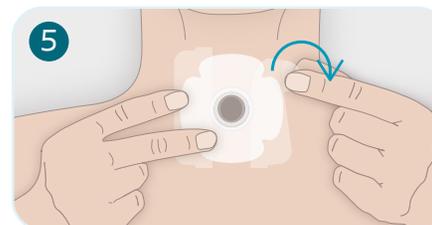

5 Remove the side protective liners.



6 Massage the adhesive onto your skin.

How to remove Provox Life Night Adhesive




Gently remove the Night Adhesive by using a wet cloth. Make sure no water drips into your stoma.

Adhesive Collection Log

Keep track of your adhesive use with this log. Take note of adhesive fit, seal, and comfort while wearing.

Tracking how your skin looks and feels after wearing an adhesive will help you understand how your skin responds, so you can identify which adhesive is the best choice for you.

Use the Adhesive Collection Reference Guide to help identify which adhesive and shape you are using.

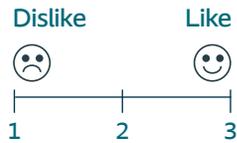

	Adhesive Used		Adhesive Shape	Wipes Used?	Rank Adhesive
	Standard, Sensitive, Stability, or Night		For Standard and Sensitive only	Yes or No	Based on fit, seal, and skin condition
1					
2					
3					
4					
5					
6					
7					

Notes: _____

HOW TO ASSESS FIT AND SEAL:

Adhesive should:

- Feel comfortable.
- Last throughout the day.
- Sit flat on the skin.
- Have no air leaks.
- Stay in place during daily activities.

HOW TO ASSESS SKIN:

- Wait 1-2 minutes after removing your adhesive to assess your skin.
- Understand that light discoloration is normal.
- Note any significant redness or irritation.

	Adhesive Used		Adhesive Shape	Wipes Used?	Rank Adhesive
	Standard Stability	Sensitive Night	For Standard and Sensitive only	Yes or No	Based on fit, seal, and skin condition
8					
9					
10					
11					
12					
13					
14					

The adhesive that works best for me is:

References:

1. Bien S, Okla S, van As-Brooks CJ, Ackerstaff AH. The effect of a Heat and Moisture Exchanger (Provox HME) on pulmonary protection after total laryngectomy: a randomized controlled study. *Eur Arch Otorhinolaryngol*. 2010;267(3):429-35.
 2. Parrilla C, Minni A, Bogaardt H, Macri GF, Battista M, Roukos R, et al. Pulmonary Rehabilitation After Total Laryngectomy: A Multicenter Time-Series Clinical Trial Evaluating the Provox XtraHME in HME-Naive Patients. *Ann Otol Rhinol Laryngol*. 2015;124(9):706-13.
 3. Beck ACC, Retel VP, Bunting G, Sethi RKV, Deschler DG, van den Brekel MWM, et al. Cost-effectiveness analysis of using the heat and moisture exchangers compared with alternative stoma covers in laryngectomy rehabilitation: US perspective. *Head Neck*. 2020;42(12):3720-34.
 4. Longobardi Y, Galli J, Di Cesare T, D'Alatri L, Settini S, Mele D, et al. Optimizing Pulmonary Outcomes After Total Laryngectomy: Crossover Study on New Heat and Moisture Exchangers. *Otolaryngol Head Neck Surg*. 2022;167(6):929-40.
- Verschuur HP, Gregor RT, Hilgers FJ, Balm AJ. The tracheostoma in relation to prosthetic voice rehabilitation. *Laryngoscope*. 1996;106(1 Pt 1):111-5.
- Nybaek H, Jemec GB. Skin problems in stoma patients. *J Eur Acad Dermatol Venereol*. 2010;24(3):249-57.
- Hoeflok J, Guy D, Allen S, St-Cyr D. A prospective multicenter evaluation of a moldable stoma skin barrier. *Ostomy Wound Manage*. 2009;55(5):62-9. 26. Leemans M. General discussi

Contact us for more information.
We are always here to help.

1 800 217 0025
info.us@atosmedical.com
www.atosmedical.us

This text does not replace or represent the complete contents of the Instructions For Use (IFU) and/or Prescriber Information for the products in this text. Therefore, before prescribing and/or using any of the products included in this text, please review the entire contents of the respective IFU and/or Prescriber Information. For IFU, please visit www.atosmedical.us

Atos Medical Inc

5000 South Towne Drive, Suite 200, New Berlin, WI 53151-7956 USA
1 800 217 0025 | info.us@atosmedical.com | www.atosmedical.us

Atos and the Atos Medical logo are trademarks of Coloplast A/S.
©2026 Coloplast A/S. All rights reserved. | MC-3399, 112025