


Provox Life Adhesives are designed for a better fit and more comfort. They include choices suitable for a variety of skin types and stoma contours, so you can find the one that's best for you.

Simple steps to **APPLY** Provox Life Adhesive.

1 Clean skin

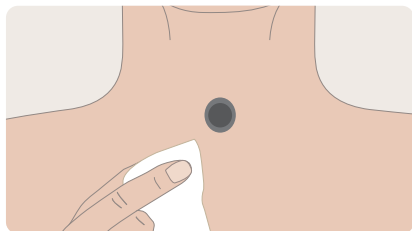


Clean the skin around your stoma with Provox Cleaning Towel, or with mild soap and water. Let dry.

Provox Cleaning Towel



Tip: Shave all hair around the area of application.

2 Prepare skin

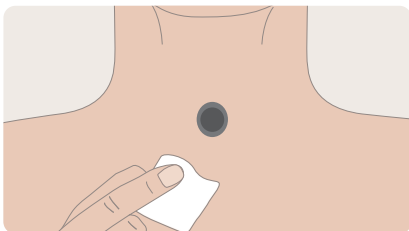

Apply Provox Skin Barrier. Let dry.

DO NOT use Skin Barrier when applying Provox Life Night Adhesive.

Provox Skin Barrier




3 Warm adhesive



When using Provox Life Sensitive Adhesive, rub the adhesive between your hands to warm it up.

4 Remove liner

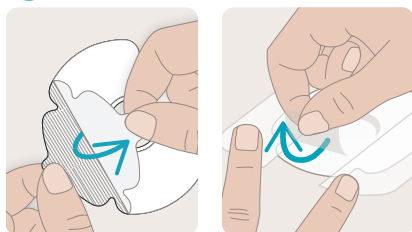


Sensitive and Standard Adhesive

Night and Stability Adhesive


Remove the protective paper.

5 Place adhesive

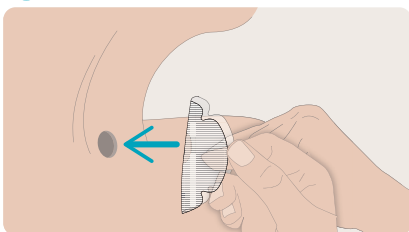

Line up the bottom ring of the adhesive with the lower lip of your stoma.

6 Massage adhesive gently

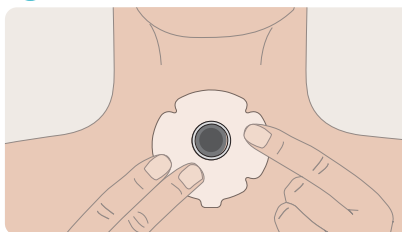

Massage the adhesive onto your skin. Start from the center and move out. Make sure there are no gaps or wrinkles.



Tips for applying an adhesive:

- Touching the sticky layer with your fingers may reduce adhesion.
- If you need stronger adhesion, use Skin Tac Wipe™ or Provox Silicone Glue sparingly.
- Attach the adhesive with the neck in a neutral position.



Skin Tac Wipe




Provox Silicone Glue

Tips for an airtight seal:

- Allow the adhesive to bond to your skin before speaking. For best results wait:
Sensitive = 15 minutes
Standard and Stability = 1 hour
- Always use two fingers to hold the adhesive in place when removing the HME.
- Remove the HME from the adhesive when you begin to cough.
- After a cough, carefully remove any mucus from the skin.
- Speaking with too much force may cause the seal to loosen.

Simple steps to **REMOVE** Provox Life Adhesive.

1 Remove adhesive

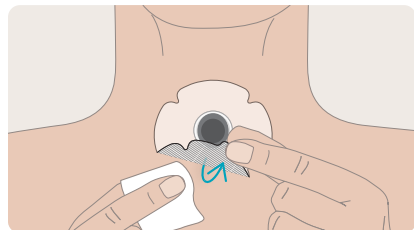

REMOVE the adhesive slowly, gently supporting the skin as you peel.

For Provox Life **Standard** Adhesive and Provox Life **Stability** Adhesive, first apply Provox Adhesive Remover on top of the adhesive.

Then lift the tab carefully and add more underneath the adhesive.

For **Sensitive**, apply Adhesive Remover or water under the adhesive.

Night Adhesive should be removed with water only.



Tips to minimize skin irritation (ex. redness):

- Use Skin Barrier wipes before putting on your adhesive.
- Change your adhesive regularly.
- DO NOT remove the adhesive without Adhesive Remover (Standard and Stability) or water (Sensitive and Night).

Scan here to watch our CareTip videos with tips and tricks for using adhesives.



www.atosmedical.us/resources/caretips

Guide to Provox Life Adhesives

Provox Life Sensitive Adhesive

Sensitive adhesive is a skin-friendly adhesive for all patients. It provides a good seal.




Provox Life Night Adhesive

Night adhesive is a soothing and comfortable adhesive for patients with irritated or breached skin.



Provox Life Standard Adhesive

Standard adhesive is a strong and flexible adhesive for patients with healthy skin.

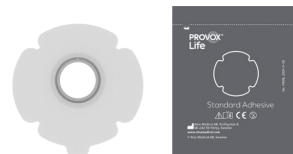

Provox Life Stability Adhesive

Stability adhesive provides firm support for patients who speak frequently, have a deep stoma or speak hands-free.



Care Essentials

Provox Adhesive Remover

To help remove adhesive

- Alcohol-free
- Makes removal of adhesive and residue easier



Provox Silicone Glue

Improves the adhesion between the skin and the adhesive.



Skin Tac Wipe

Provides a tacky barrier between adhesive and skin.



Provox Cleaning Towel

To clean the skin before putting on an adhesive.

- Cleans and removes oil from the skin
- Alcohol-based and unscented



Provox Skin Barrier

To leave a protective layer on the skin.

- Alcohol-free
- Use after cleaning the skin and before applying an adhesive
- May help improve adhesive seal



This text does not replace or represent the complete contents of the Instructions For Use (IFU) and/or Prescriber Information for the products in this text. Therefore, before prescribing and/or using any of the products included in this text, please review the entire contents of the respective IFU and/or Prescriber Information. For IFU, please visit www.atosmedical.us

Atos Medical Inc

5000 South Towne Drive, Suite 200, New Berlin, WI 53151-7956 USA | 1 800 217 0025 | info.us@atosmedical.com | www.atosmedical.us

Atos and the Atos logo are trademarks of Coloplast A/S. ©2026-03 Coloplast A/S. All rights reserved. MC-2713, 031326