PROVOX[®]

Fenestration Punch

Rx
ONLY

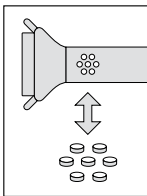
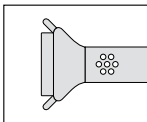
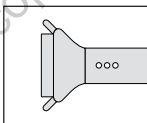
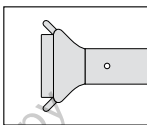
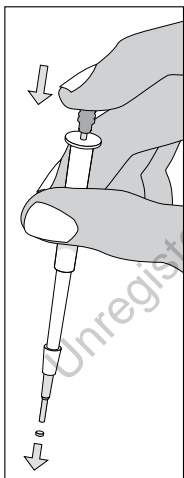
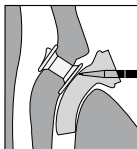
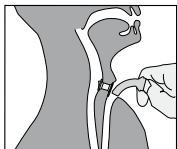
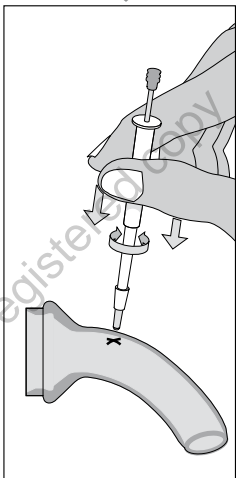
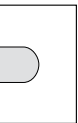
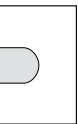
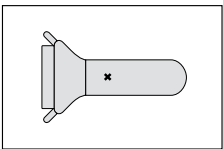
CE

ATOS

MEDICAL

Leading the way in ENT

Prescription information

CAUTION: United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

Disclaimer

Atos Medical offers no warranty - neither expressed nor implied - to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox® is a registered trademark owned by Atos Medical AB, Sweden and Provox LaryTube™ is a trademark owned by Atos Medical AB, Sweden.

Contents

ENGLISH	6
DEUTSCH	9
NEDERLANDS.....	13
FRANÇAIS	16
ITALIANO.....	19
ESPAÑOL	22
PORTUGUÊS.....	26
SVENSKA	30
DANSK.....	33
NORSK.....	36
SUOMI	39
ČESKY	42
MAGYAR.....	45
POLSKI.....	48
ΕΛΛΗΝΙΚΑ	51
TÜRKÇE	55
РУССКИЙ	58
日本語版.....	62
한국어	65
ORDERING INFORMATION	68

The manual, which accompanies this product, may be revised from time to time and must therefore be reviewed prior to each procedure in which the product is used.

Intended use

The Fenestration Punch is used for making small fenestrations in a Provox LaryTube at desired locations.

Device description

The Fenestration Punch is made of polypropylene, stainless steel and silicone and is used for making small fenestrations in a Provox LaryTube. This is done when the Provox LaryTube is intended to be used in combination with a voice prosthesis.

Warning

- To avoid cross contamination from one patient to another, use appropriate hygienic procedures and make sure that each used device is cleaned, disinfected, dried and steam sterilized.

- Disassemble the Punch, Punch cap and Ejector bar prior to cleaning and sterilization.

Precautions

- Make sure that no silicone residues are left in the holes or lumen of the LaryTube after fenestration.
- Cleaning, disinfection and sterilization are described in “Instructions for cleaning and sterilization”.

Cleaning and sterilization

Disassemble the Punch, Punch cap and Ejector bar prior to cleaning and sterilization. Cleaning, disinfection and sterilization are described in the “Instructions for cleaning and sterilization”. If this information is missing, please contact your supplier or visit our website on www.atosmedical.com.

CAUTION: Always ensure mechanical integrity before each use.

How supplied

The device is manufactured in a cleanroom and supplied non-sterile.

Disposal

After use, the product may be a potential biohazard. Handling and disposing should be carried out in agreement with medical practice and applicable national laws and legislations.

Ordering information

See last page of this manual.

Date of printing

See version number on the back cover of this manual.

User assistance information

For additional help or information please see back cover of this manual for contact information.

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© Atos Medical AB, Sweden. 201202A, REF 90735