

## Is Your Voice Prosthesis Leaking?

Do you suffer from:

1. Coughing during or after eating/drinking?
2. Food/Fluid stained mucus?
3. Increased secretions?
4. Recurrent chest infections?

If you answered yes to any of these questions, your voice prosthesis might be leaking.



### LEAKAGE TEST

1. Stand in front of a mirror with a good light and clean your Provox voice prosthesis.
2. Take a sip of something colored (e.g. milk or coffee, as this is easier to see).
3. Look to see if anything leaks through or around the voice prosthesis. If you have a hard time seeing your voice prosthesis, ask someone else to look for you.

