


## Bedside Instructions for Use of Provox® Laryngectomy Pulmonary Kits

**ALWAYS consult the Instruction For Use for EACH product for complete details.** A comprehensive Atos Medical Laryngectomy Pulmonary Kit In-Service is available in digital format.


### How to place the Provox® LaryTube™

- Moisten the tube with water. If needed, use medical grade water soluble lubricant and gently slide the Provox LaryTube into the stoma.

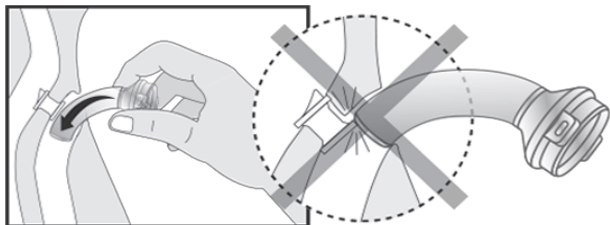

### When to use the Provox® Heat and Moisture Exchanger (HME) Cassette

- Use the HME 24/7, even while sleeping
- **DO NOT** rinse out the HME Cassette

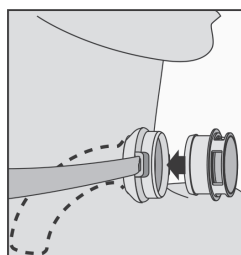


### When to clean the Provox LaryTube

- Exact frequency is patient dependent
- Several times a day, whenever dirty
- When mucus collects in the tube
- If patient becomes short of breath

### How to clean the Provox LaryTube

- Gently remove the tube
- Scrub thoroughly with the Provox® TubeBrush in a hot water-soap solution
- Do not use oil-based soaps
- Rinse copiously in hot water and follow with a thorough rinse in distilled water

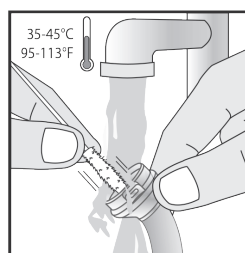

### When to change the Provox HME cassette

- Change the HME at least every 24 hours and more frequently as needed
- Change the HME when patient complains about increased breathing resistance, which may occur if the HME becomes saturated with mucus

**NOTE: Provox HME Cassettes can be used while the patient is sleeping.**

**DO NOT PLACE EXTERNAL HUMIDIFIER OVER HME!**

**Provox<sup>®</sup>**  
**Heat and Moisture Exchanger**  
**(HME) In Use**

**Important!**

- Change the HME AT LEAST every 24 hours or more often as needed
- Use HME 24/7, even while sleeping
- DO NOT rinse out the HME Cassette

# Provox® Heat and Moisture Exchanger (HME) Log

## Important!

- Change the HME AT LEAST every 24 hours or more often as needed
- Use HME 24/7, even while sleeping
- DO NOT rinse out the HME Cassette

Post-Op Day (POD)	Date	Time of HME Change	RT/RN Initials	Time of HME Change	RT/RN Initials	Time of HME Change	RT/RN Initials
POD 1							
POD 2							
POD 3							
POD 4							
POD 5							
POD 6							
POD 7							
POD 8							
POD 9							
POD 10							
POD 11							
POD 12							
POD 13							
POD 14							
POD 15							
POD 16							

© Atos Medical AB, MCI298-CoUSENL\_201805