

Is Your Voice Prosthesis Leaking?

Do you suffer from:

1. Coughing during or after eating/drinking?
2. Food/Fluid stained mucus?
3. Increased secretions?
4. Recurrent chest infections?

If you answered yes to any of these questions, your voice prosthesis might be leaking.



LEAKAGE TEST

1. Stand in front of a mirror with a good light and clean your Provox voice prosthesis.
2. Take a sip of something colored (e.g. milk or coffee, as this is easier to see).
3. Look to see if anything leaks through or around the voice prosthesis. If you have a hard time seeing your voice prosthesis, ask someone else to look for you.

